

ABL800 Flex Plus Blood Gas Training Record

FOR-POC-034

Issued: 8 November 2023

Approved by: J. Paynter

First Name:	Surname:	Employee No:
Hospital:	Department:	Position:
Email:	Date:	
Training Type ☒ Initial/New ☐ Retraining ☐ Other (please specify)		
Training Outcome/Competency achieved		Yes ☒ N/A ☐
Pre-Analytical		
Aware of local Safety Policy for Blood Collection and Handling		
Can identify all the components of the ABL800Flex Plus Analyser: Touchscreen, Scanner, FLEXQ tray (if applicable), Printer, Syringe/Capillary inlet flap, Rinse and Waste bottle,		
Understands the traffic light system for Analyser status machine		
Aware of the automated QC and Calibration process and lockout of analytes if QC/Cal unacceptable		
Aware of the sample requirements – Li Heparin/Balanced Heparin syringe, well mixed and filled correctly		
Knows the sample types that can be tested Venous, Arterial, Cord Blood, Capillary and sample volume required		
Knows the time frame that is acceptable to run sample after collection and how to fill/prepare syringe correctly		
Analytical		
Acknowledges the requirements for correct patient identification process at bedside and on device before testing		
Knows the importance of entering Patient Detail correctly and potential consequences if they are incorrect		
Can locate the POCT Patient Reconciliation Form and is aware when this form must be used		
Can mix and run patient syringe sample (If FLEXQ tray present, follow LWI-POC-013)		
Acknowledge the importance of correct placement of the syringe on FLEXQ tray to avoid bending probe		
Can mix and run patient capillary sample		
Acknowledges responsibility for all testing done under your Login and will not share Login details		
Post Analytical		
Can locate reference ranges, significant and critical limits and is aware to notify clinician to these results		
Is aware to repeat tests with a new collection if results are unexpected and do not fit with patient's clinical status		
Can reprint results if required		
Can recall troubleshooting pathways and knows resources available and who to call for assistance		
Maintenance		
Can describe the appropriate cleaning/disinfecting procedure if required		
Can replace the paper roll in the printer, Rinse Bottle and Waste Bottle as required.		
Superuser		
Weekly: Refill QC (Autocheck) Carousel & Protein removal		
As Required: Cal1, Cal2, Clean solutions, Gas1, Gas2, Inlet gasket, Inlet Probe		
Scheduled: Membrane changes, perform tHb Calibration (3 months)		
Can run QAP samples		
Operator Declaration – Please read before signing		
I have completed my ABL800 Flex Plus training and am aware of my responsibility as a Point of Care Operator. I know where to locate the procedures and resources for the ABL800 device and will follow the correct procedures when using this device. I know it is my responsibility to action critical results from the ABL800 device and I am responsible for testing done under my Operator ID.		
I will not share my Operator ID		Operator Signature:
POCT / Trainer to Complete		
Operator to be registered as: Basic User ☐ Superuser ☐ Trainer ☐		
Trainer Name:		Trainer Signature:

Send completed forms to: poc@austin.org.au